



Description

Chamber X-Treme is a concentrated blend of inorganic acids designed to remove heavy scale from chambers and boilers. This product is primarily designed for use inside washer-disinfector, cart washer, and sterilizer chambers, and for use in descaling boilers. Chamber X-Treme may also be used in manual applications.

Features

- Removes hard water scale and other sediments from boilers
- Removes excessive carbonate buildup from inside machine chambers

Directions for Use

Chamber X-Treme should only be used in chambers warm to the touch or cooler. This eliminates flash evaporation of the product, preventing long term corrosion. This product should not remain on metal surfaces longer than 20 minutes. If allowed to do so, discoloration, corrosion, tarnishing, and/or pitting may occur.

Depending on the severity of the scale buildup, **Chamber X-Treme** may be used at full concentrate or may be diluted up to 5 to 1 with a pure water source, i.e. reverse osmosis or deionized water. If reverse osmosis or deionized water are not available, tap water with chloride concentrations less than 50 ppm may be used. When diluting, to prevent any undesirable reactions, it is important that product be added directly into the water, rather than adding water into undiluted product. For more severe buildups, rinse and re-application may be required. Nylon brushes and cotton rags may also be required. Be sure to adequately rinse when cleaning is complete.

Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

DO NOT ADD BLEACH or any other chemical to **Chamber X-Treme**. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir. Store below 120°F (49°C).

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



Chamber X-Treme

Concentrated Manual Chamber Cleaner
and Boiler Descaler

Properties

Color: Blue liquid

Odor: Acidic odor

Product pH: <1.0

Store below 120°F (49°C)

Use up to 35 months from
date of manufacture

Package Sizes Available

4x1 Gallon

FICCB-CP-04X1-8401



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

