



Description

CMD-Zyme is a free-rinsing, dual-enzyme detergent, designed to break down organic surgical soils. It breaks down blood, tissue, fats, oils, and grease in washer-disinfectors, ultrasonics, and manual applications.

Features

- Multifunctional; can be used in sink or washers
- Dual-enzyme blend targets a variety of soils including blood, tissue, fats, and other soils
- Compatible with most Sterile Processing Department substrates
- Effective in varying hardness and temperatures

Directions for Use

Washer-Disinfectors Applications: Dose CMD-Zyme between 0.25-2.0 oz./gal. of water (2.0-16 mL/L), according to water quality and typical soil load. Consult the machine manufacturer's directions for use of the specific machine being used.

Ultrasonic Applications: Dose CMD-Zyme between 1.0-4.0 oz./gal. of water (8.0-32 mL/L), according to water quality and typical soil load. Consult the ultrasonic manufacturer's directions for use of the specific model being used.

Sink (Manual) Applications: Dose CMD-Zyme between 1.0-4.0 oz./gal. of water (8.0-32 mL/L). Soak duration should be determined by the individual facility's standard operating procedures. Intercon recommends a minimum soak time of 2 minutes.

Endoscopes: Dose CMD-Zyme between 0.50-2.0 oz./gal. of water (4.0-16 mL/L). Process the soiled scopes according to the manufacturer's IFU or specifications determined by the individual facility's standard operating procedures.

Store detergent at temperatures below 100°F (38°C). The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir. Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.

Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to CMD-Zyme. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



CMD-Zyme

Dual-Enzymatic Detergent for Sterile Processing

Properties

Color: Purple liquid

Odor: Sassafras

Product pH: between 6-8.5

Store below 100°F (38°C)

Use up to 23 months from date of manufacture

Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-8801
2.5 Gallon	FICCB-CP-2.5G-8801
5 Gallon	FICCB-CP-005T-8801
15 Gallon	FICCB-CP-0015-8801



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2026 Intercon Chemical Company.

