



## Description

**Opti Spray** is a ready-to-use, neutral pH, non-enzymatic point of use spray designed as the first step in sterile reprocessing for operating room, emergency room, or ophthalmic suite applications. Opti Spray is compatible with virtually every substrate used in hospitals today.

## Features

- High performance cleaning without enzyme or alkalinity
- Non-corrosive to aluminum, stainless steel, plastics, glassware, and most other Sterile Processing Department (SPD) substrates
- Excellent in varying water conditions
- Free rinsing

## Directions for Use

Opti Spray is for point of use applications only. **Do NOT use for automatic washing applications.** Apply Opti Spray directly onto soiled instruments as soon after the procedure as possible. Application volume should be directly reflective of soil load. Hold spray tip 6 to 12 inches away from instruments while spraying for maximum coverage. Amply apply to delay dry down. Rinse instruments or other substrates thoroughly before continuing processing.

## Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use.

Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to Opti Spray. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



# Opti Spray

Ready-To-Use Neutral pH, Non-Enzymatic Point of Use Spray  
for Everyday and Ophthalmic Sterile Processing

## Properties

**Color:** Colorless to light yellow liquid

**Odor:** Mild

**Product pH:** 6.0-8.0

Store below 120°F (49°C)

Use up to 23 months from date of manufacture

## Package Sizes Available

12x32 oz.

FICCB-CP-1232-8810



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • [interconchemical.com](http://interconchemical.com)

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

